

EZsync USB cable installation as simple as 1-2-3-4

Legal disclaimer: This cable is not designed or manufactured by onetouch or lifescan

Contact ezsyncable@gmail.com for customer service, you will have a response within 24 hours

Step 1: Open this document online

The latest version of this document is saved online at: <http://tiny.cc/onetouchFT>(case sensitive). **Please open this document online** so you can click the links below from your web browser directly which is a more convenient way to download the files. Of course you can also directly type in the weblinks below in your web browser to access the files. Please be reminded these links are **case sensitive**.

Step 2: Download and install driver software

Go to one of the web links listed below, **install the USB driver for the cable** at no charge. Most likely for windows7, 8 and 10, the cable is “plug and play” so it doesn’t require any USB driver installation.

For windows, type: <http://tiny.cc/ftdiwin> (case sensitive) in your web browser to download the driver

For Mac OS X 10.3-10.8, type: <http://tiny.cc/ftdimac2>(case sensitive) in your web browser to download the driver

For Mac OS X 10.9 or above, type: <http://tiny.cc/ftdimac1>(case sensitive) in your web browser to download the driver

To assist your driver installation, please refer to EZsync Cable Driver Installation Guideline saved online at: <http://tiny.cc/InstructionFTDI>(case sensitive)

While this is optional, but you can always go the following page to look for the **latest USB drivers** for the installation: <http://www.ftdichip.com/Drivers/VCP.htm>

Step 3: Install diabetes management software

Go to Onetouch™ website to **install OneTouch® Reveal Web app** at no charge. You should sign up the onetouch web app at: <http://tiny.cc/RevealSignup> (Case sensitive) first, then you can download the **data transfer tool** on the “help page” after sign in.

Step 4: Connect the cable all the way into the Meter

Connect the meter to the PC. Plug USB end of the EZsync USB Cable into the USB port on your PC. Plug the other end of the EZsync USB Cable **all the way** into your meter. Please refer to the onetouch reveal web app manual here at: <http://tiny.cc/onetouchreveal>(case sensitive).

I appreciate if you could **contact us before requesting cable return** if you have any technical issue to get the cable to work properly with your meter. we will do all we can to solve the issues as soon as possible. Thank you!! Please send email to : support@purenitetechnology.com and let us know your meter brand, computer OS(Mac or windows)and cable part number(sticker on USB end of the cable).